

Elite

SEAT RANGE

From walking stick to restful seat in an instant!

Thank you for choosing a Linden Elite seat. We wish you many years of happy use. For your complete safety please read all instructions:

- **Elite Stroller** – Walking height: 825mm (32¹/₂"")
Seat height: 755mm (29³/₄"")
- **Elite Walker & Tracker** – Walking height: 855mm (33³/₄"") to 935mm (36³/₄"")
Seat height: 775mm (30¹/₂"") to 850mm (33¹/₂"")
- **Elite Packaway** – Walking height: 815mm (32"") to 925mm (36¹/₂"")
Seat height: 735mm (29"") to 815mm (32"")
- **Maximum user weight:** 125kg

Your Elite seat is intended for use only as a one person seat. Do not use it for anything else.

CAUTION: Single tube seatsticks are for those who can balance safely on the seat and form a stable tripod with their weight shared evenly between seat & feet. If you have a disability or condition which may affect your balance, always seek medical advice before use.

1. Where applicable, height is adjusted by pressing in both spring buttons and sliding the foot to the desired length. Ensure both buttons are locked in a set of holes before using.
2. To open the seat: Grasp the black locking sleeve and pull down firmly.
3. Tip-up the seat into a horizontal position. Ensure that the locking sleeve is fully engaged with the seat.
4. Place the seat behind you with the base approx. 35cm (14"") away. Lean the seat towards you and sit slowly with your feet approx. 35cm (14"") apart.



PACKAWAY MODEL

Firstly remove the two feet from the bag. Then remove the seat being sure to hold both seat and tubes as you do so. Ease tubes out so that they slot together.



MODELS AVAILABLE

Stroller
(non adjustable, rubber ferrule)

Walker
(adjustable, rubber ferrule)

Tracker
(adjustable, point & plate for soft ground only)

Packaway
(adjustable, rubber ferrule & point & plate)

COLOURS AVAILABLE
Black or Green

